### ARGON MEDICAL DEVICES, INC.

1445 Flat Creek Road  
Athens, Texas 75751 USA  
Tel: 800-927-4669  
Tel: +1 903-675-9321  
www.argonmedical.com

### EMERGO EUROPE

Prinsessegracht 20  
2514 AP The Hague  
The Netherlands  
+31 70 345 8570

### Argon Medical Devices UK Ltd

Eastgate Business Centre  
Eastern Avenue  
Burton-on-Trent  
Staffordshire  
DE13 0AT

### Intended Use/Purpose

Skater™ Centesis Catheters are intended for small volume drainage procedures and percutaneous fluid aspirations.

### Device Description

Skater™ Centesis Catheters are intended for small volume drainage procedures and percutaneous fluid aspirations. The catheter has four side holes to facilitate drainage within small cavities but is also offered without side holes. The catheter is attached to the hollow stylet via rotating Luer lock that permits aspiration of fluid prior to stylet removal. The Luer lock allows for one-handed placement. The catheter has a polyurethane body and hub which allows for the stainless-steel cannula to easily pass through. This device is designed to work in conjunction with aspiration syringes.

The Skater™ Centesis are available in the following sizes:

French Size	Needle O.D.	Catheter O.D.	Catheter I.D.
4	20 ga (0.9 mm)	1.4 mm	1.0 mm
5	19 ga (1.1 mm)	1.7 mm	1.2 mm

### Indication for use

The product is for percutaneous fluid drainage.

### Duration

Transient, less than 60 minutes.

### Warnings

- This device was designed, tested and manufactured for single use only. Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury. Do not reuse, reprocess or re-sterilize this device.
- Inspect the package integrity before use.
- Do not use if package appears open or if the expiry date has been exceeded.
- Do not continue to use if any of the component are damaged during the procedure.
- The product must only be used by qualified personnel who are familiar with the technique.
- During insertion avoid contact with bone, cartilage and scar tissue which can damage the catheter tip.

### Preparation/Procedure

(Single-step catheterization through a skin incision)

- Perform the skin incision under local anesthesia.
- Introduce the SKATER™ Centesis Catheter into the cavity using ultrasound guidance, CT or fluoroscopy.
- Remove the needle while holding the catheter in position.
- Connect a syringe or a suction tube to the catheter hub.

### Storage

Store at controlled room temperature. Avoid extended exposure to light.

### Disposal

After use, this product may be a potential biohazard. Dispose in accordance with applicable laws and regulations.

**NOTE:** In the event a serious incident related to this device occurs, the event should be reported to Argon Medical at [quality.regulatory@argonmedical.com](mailto:quality.regulatory@argonmedical.com) as well as to the competent health authority where the user/patient resides.

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