 **ARGON MEDICAL DEVICES, INC.**  
1445 Flat Creek Road  
Athens, Texas 75751 USA  
Tel: +1 (903) 675 9321  
Tel: +1 (800) 927 4669  
www.argonmedical.com

**EC** **REP** **EMERGO EUROPE**  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands  
+31 70 345 8570

 The symbols glossary is located electronically at [www.argonmedical.com/symbols](http://www.argonmedical.com/symbols)

IFU 20-2007-43/0819B

## En - English

### Intended Use

The device is designed to provide a percutaneous conduit for the atraumatic passage of other intravascular devices into the vasculature.

### Warnings

- Dilators are intended for use by trained medical personnel.
- This device was designed, tested and manufactured for single use only. Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury.
- Do not use if package appear open or if the expiry date has been exceeded.

### Procedure

1. Using aseptic technique, establish access to the vessel.
2. Advance a guidewire into the vessel.
3. Remove any components remaining over the guidewire, leaving only the guidewire in position.
4. Inspect the dilator for damage before use.
5. Flush the dilator lumen with sterile isotonic saline prior to use.
6. Thread the vessel dilator over the guidewire and advance the dilator into the vessel. Slightly rotate the dilator as it enters the access site and enters the vessel.
7. Remove the dilator leaving the guidewire in place.

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