
ARGON MEDICAL DEVICES, INC.
1445 Flat Creek Road, Athens, Texas 75751 USA
Tel: 800-927-4669;
Tel: +1 903-675-9321
www.argonmedical.com

STERILE EO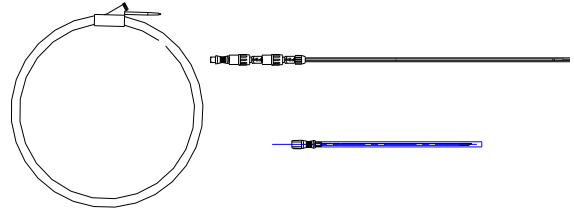

ENGLISH

Caution: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

SKATER™ Introducer System with Radiopaque Marker

Intended Use: SKATER™ Introducer Systems with Radiopaque Marker facilitate introduction and placement of a guidewire.

Contraindications: Physician judgment should be used in determining for which patients percutaneous interventional procedures are contraindicated.

Warning: The product must only be used by qualified personnel who are familiar with the technique. **Please note:** Loosen the straightener together with the guidewire on the circular dispenser.

Procedure:

1. Perform a skin incision under local anesthesia.
2. Do a fine needle puncture using ultrasound or X-ray.
3. Remove the stylet and introduce the 0.018" guidewire.
4. Remove the needle and dilate with the fully assembled coaxial sheath/dilator.
5. Loosen the dilator (4Fr) checking that the tip of the outer sheath (6Fr) has been properly placed inside the cavity (note the gold ring marker).
6. Remove the stiffening cannula and 4Fr dilator.
7. Introduce the 0.038" guidewire.
8. Remove the outer 6Fr sheath and withdraw the 0.018" guidewire.
9. Advance the required catheter over the guidewire. (Further dilating may be necessary).

Storage: Store in a cool, dry area.

Disposal: After use, this product may be a potential biohazard. Dispose in accordance with applicable laws and regulations.

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